

# **SPEAKMAN COMPANY**

## **VS-2054 Installation, Maintenance & Operation Instructions**

### **ADA / HANDICAP SHOWER AND BAR**

#### **DESCRIPTION**

Speakman ADA handicap shower system includes hand held shower (VS-100-AF), 5' stainless steel hose (G63-0060), swivel connection (VS-120), vacuum breaker (VS-151), and a 24" grab bar (VS-123). Connects up to standard shower arm. Handheld shower puts out 2.5 GPM/9.46 LPM to meet ANSI A112.18.1 Standard. ADA Compliant.

#### **SPECIFICATIONS**

SUPPLY: 1/2" NPT FEMALE  
SHIPPING WEIGHT: 3.3 LBS.  
FLOW RATE: 2.5 GPM/9.46 LPM, MAX.,  
HAND HELD, VARIABLE

#### **INSTALLATION**


Remove (turn counterclockwise) your existing showerhead from the shower arm. Clean the shower arm threads of any debris and wrap teflon tape in the clockwise direction around shower arm threads. Assemble vacuum breaker to shower arm. Hand tighten. Ensure washer is inside connection to prevent leaks. Assemble one end of the hose to the vacuum breaker. Hand tighten firmly. Assemble the hand held showerhead to the swivel. Then connect the other end of the swivel to the hose with a washer. When installing the bar the location is optional but restricted by the hose length. Attach bar to wall using fasteners supplied. Slip the hand held swivel connector onto post. Adjust height of post by loosening post handle. Tighten handle to lock into place.

#### **OPERATION**


To operate the handheld shower, push the pause button to one side to bring the flow down to a trickle, then simply push it to the other side to turn the flow back on full force.

#### **MAINTENANCE**

The finish of your SPEAKMAN VERSATILE® hand held shower and all of the other components can be cleaned by using mild soap and warm water. Dry immediately with a soft, clean cloth for best results. The flow control device limits water flow to 2.5 gallons per minute and is located in the base of the handheld. The small orifice in the flow regulator, resulting in lower flow and a weaker performance, may become clogged. To clean the flow regulator, simply remove the hose from the hand shower and clean as necessary.

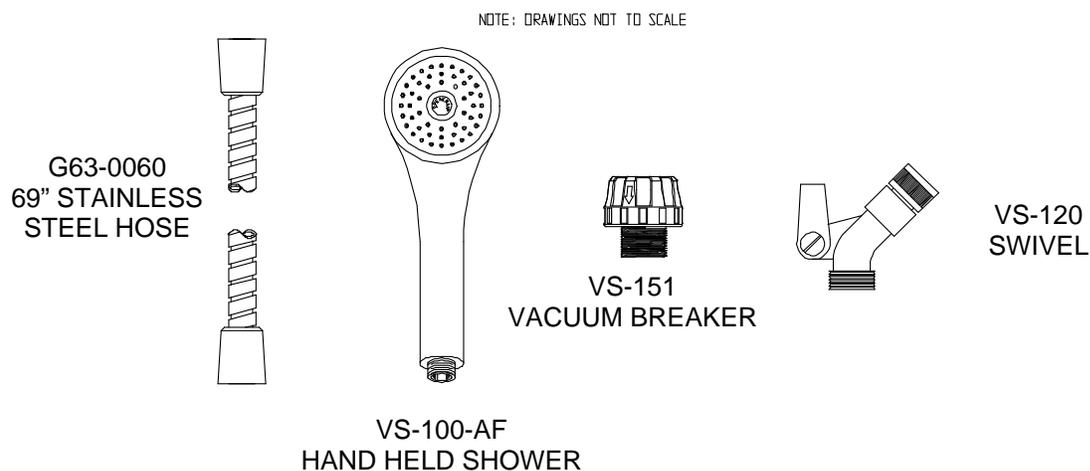

## ROUGH-IN MEASUREMENTS

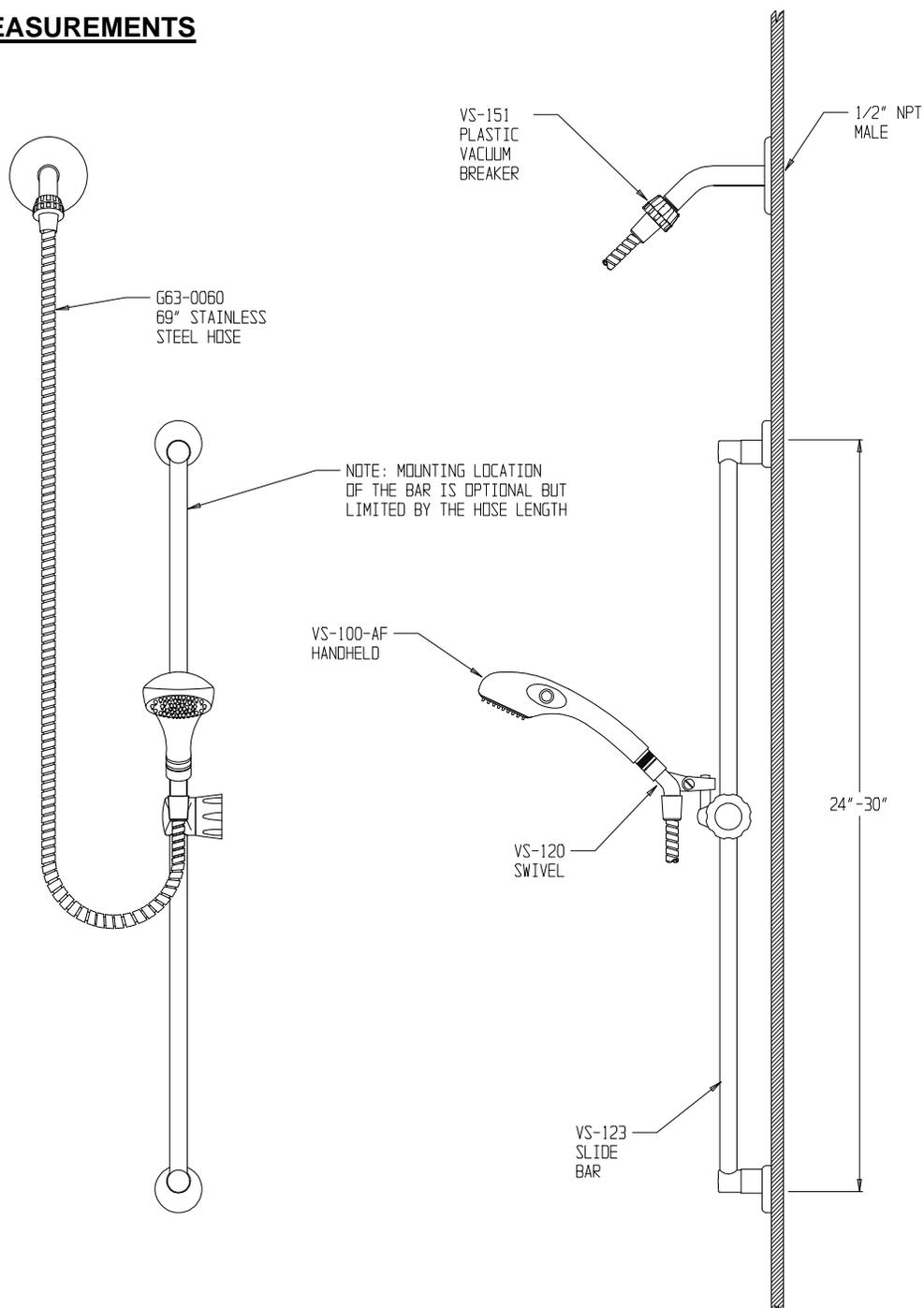

NOTE:

1. ALL DIMENSIONS ARE IN INCHES (MILLIMETERS UNLESS OTHERWISE SPECIFIED AND ARE SUBJECT TO CHANGE WITHOUT NOTICE.
2. UNLESS OTHERWISE SPECIFIED, ALL INLETS AND OUTLETS ARE 1/2" NPT FEMALE.
3. THE LOCATION OF THE POST AND SUPPLY ELL IS OPTIONAL, BUT RESTRICTED BY THE 69" HOSE LENGTH RUNNING FROM THE SUPPLY ELL CONNECTION TO HAND SHOWER

**SPEAKMAN®**

PO Box 191 Wilmington, DE 19899 Phone: 800-537-2107 Fax: 800-977-2747  
Visit us on the web at [www.speakmancompany.com](http://www.speakmancompany.com)

10/07  
92-2939